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Minimal-traumatic axial tooth extraction with ZALEX®

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During and after the tooth extraction, there are frequently complications. They are correlated with the burden in the tooth and the surrounding tissues during the luxation and contribute to tissue loss. ZALEX® is an extraction system that optimally uses the force acting in the longitudinal axis of the tooth, so that minimally luxation and rotation movements are necessary. Methodically and instrumentally ZALEX® follows on seamlessly after the actual tooth extraction, allowing easy acceptance into the practice process.

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Connection between holding forceps (item no. 310-73) on the right and lifting forceps (item no. 300-00) on the left for axial extraction of tooth 37.

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The holding forceps (item no. 310-73) is applied on tooth 37. The lifting forceps (item no. 300-00) lies with absorbed support (item no. 300-90) on teeth 36, 35, 34 and 33.

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The lifting part of the lifting forceps (item no. 300-00) is connected with the peak of the working part of the holding forceps (item no. 310-73).

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The working parts of the lifting forceps (item no. 300-00) are splayed; tooth 37 is axially extracted.

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Significant axial extraction progress.

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Result: minimal-traumatically extraction trauma after ZALEX-extraction

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